

Reporting Title: Dog Dander Component Reflex, S**Performing Location:** Rochester**Ordering Guidance:**

For a listing of allergens available for testing, see Allergens - Immunoglobulin E (IgE) Antibodies

Specimen Requirements:

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions: Centrifuge and aliquot serum into a plastic vial.

Specimen Minimum Volume:

0.7 mL

Forms:

If not ordering electronically, complete, print, and send an Allergen Test Request (T236) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Frozen	90 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
DOGD1	Dog Dander, IgE, S	Numeric	kU/L	6098-8

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

86003

Reflex Tests:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
DOGPR	Dog Dander Components, IgE, S			No	No

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
DOGPR	INDOG	Dog IgE Antibody Interpretation	Alphanumeric		69048-7
DOGPR	CANF1	Can f 1 (Dog Dander), IgE, S	Numeric	kU/L	58773-3
DOGPR	CANF2	Can f 2 (Dog Dander), IgE, S	Numeric	kU/L	58772-5
DOGPR	CANF3	Can f 3 (Dog Serum Albumin), IgE, S	Numeric	kU/L	19738-4
DOGPR	CANF4	Can f 4 (Dog Dander), IgE, S	Numeric	kU/L	82543-0
DOGPR	CANF5	Can f 5 (Dog Dander), IgE, S	Numeric	kU/L	64973-1
DOGPR	CANF6	Can f 6 (Dog Dander), IgE, S	Numeric	kU/L	94277-1

Reference Values:

Class	IgE kU/L	Interpretation
0	<0.10	Negative
0/1	0.10-0.34	Borderline/equivocal
1	0.35-0.69	Equivocal
2	0.70-3.49	Positive
3	3.50-17.4	Positive
4	17.5-49.9	Strongly positive
5	50.0-99.9	Strongly positive
6	> or =100	Strongly positive

Concentrations of 0.70 kU/L or more (class 2 and above) will flag as abnormally high.

Reference values apply to all ages.