

C1 Esterase Inhibitor, Functional, Serum

# **Reporting Title:** C1 Esterase Inhib, Functional, S **Performing Location:** Rochester

#### Specimen Requirements:

Patient Preparation: Fasting, preferred but not required
Supplies: Sarstedt 5 mL Aliquot Tube (T914)
Collection Container/Tube:
Preferred: Red top
Acceptable: Serum gel
Submission Container/Tube: Plastic vial
Specimen Volume: 1 mL
Collection Instructions:
1. Immediately after specimen collection, place the tube on wet ice.
2. Centrifuge at 4 degrees C and aliquot serum into a 5-mL plastic vial.
3. Freeze specimen within 30 minutes.

#### **Specimen Minimum Volume:**

0.4 mL

Specimen Type	Temperature	Time	Special Container
Serum Red	Frozen	28 days	

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
C1INF	C1 Esterase Inhib, Functional, S	Numeric	%	48494-9

LOINC and CPT codes are provided by the performing laboratory.

# Supplemental Report:

No

# **CPT Code Information:**

83520



# **Test Definition: C1INF**

C1 Esterase Inhibitor, Functional, Serum

## **Reference Values:**

>67% normal (normal) 41-67% normal (equivocal) <41% normal (abnormal)</p>