

Test Definition: RMPQU

Monoclonal Protein Quantitation, Random, Urine

Reporting Title: M-protein Quantitation, Random, U

Performing Location: Rochester

Ordering Guidance:

The use of a random urine specimen is sufficient for identifying the presence or absence of monoclonal proteins, but a 24-hour specimen is preferred for quantitating and monitoring the abnormality. See SMPU / Monoclonal Protein Screen, 24 hour, Urine.

Shipping Instructions:

Refrigerate specimen after collection and send refrigerated.

Specimen Requirements:

Supplies: Urine Container, 60 mL (T313)

Submission Container/Tube: Plastic, 60-mL urine bottle

Specimen Volume: 50 mL Collection Instructions:

1. Collect random urine specimen.

2. Aliquot between 30 mL and 50 mL of urine into a plastic, 60-mL urine bottle.

Specimen Minimum Volume:

30 mL

Forms:

If not ordering electronically, complete, print, and send a Renal Diagnostics Test Request (T830) with the specimen.

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Urine | Refrigerated (preferred) | 14 days | |
| | Frozen | 5 days | |
| | Ambient | 24 hours | |

Test Definition: RMPQU

Monoclonal Protein Quantitation, Random, Urine

Result Codes:

| Result ID | Reporting Name | Туре | Unit | LOINC® |
|-----------|---------------------------|--------------|-----------|---------|
| 607975 | Albumin | Alphanumeric | mg/dL | 6942-7 |
| 607976 | Alpha-1 globulin | Alphanumeric | mg/dL | 9734-5 |
| 607977 | Alpha-2 globulin | Alphanumeric | mg/dL | 38190-5 |
| 607978 | Beta globulin | Alphanumeric | mg/dL | 9744-4 |
| 607979 | Gamma globulin | Alphanumeric | mg/dL | 9745-1 |
| 33044 | A/G Ratio | Alphanumeric | | 44293-9 |
| 33045 | M spike | Numeric | mg/dL | 40661-1 |
| 33046 | M spike | Numeric | mg/dL | 40661-1 |
| 33047 | Impression | Alphanumeric | | 49299-1 |
| PTCN1 | Protein, Total, Random, U | Numeric | ric mg/dL | |
| | Also used by tests: RPTU2 | | | |
| CRTR1 | Creatinine, Random, U | Numeric | mg/dL | 2161-8 |
| | Also used by tests: RPTU2 | | | |
| PCRT1 | Protein/Creatinine Ratio | Numeric | mg/mg | 2890-2 |
| | Also used by tests: RPTU2 | | | |

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

Components:

| Test ID | Reporting Name | CPT Units | CPT Code | Always Performed | Orderable Separately |
|---------|-------------------------------------|-----------|----------|---------------------|-------------------------|
| RPEU | Protein Electrophoresis, Random, U | | | Yes | No |
| RPTU2 | Protein/Creatinine Ratio, Random, U | | | Yes | Yes (RPTU1) |

CPT Code Information:

84156 82570 84166

0077U (if appropriate)



Test Definition: RMPQU

Monoclonal Protein Quantitation, Random, Urine

Reflex Tests:

| Test ID | Reporting Name | CPT Units | CPT Code | Always Performed | Orderable Separately |
|---------|-------------------------------|-----------|----------|---------------------|-------------------------|
| MPTRU | M-protein Mass-Fix, Random, U | | | No | No |

Result Codes for Reflex Tests:

| Test ID | Result ID | Reporting Name | Туре | Unit | LOINC® |
|---------|-----------|--------------------------------------|--------------|------|--------------------|
| MPTRU | 617119 | M-protein Isotype MS, Random, U | Alphanumeric | | In Process |
| MPTRU | 617120 | Flag M-protein Isotype MS, Random, U | Alphanumeric | | No LOINC Needed |

Reference Values:

CREATININE:

> or =18 years old: 16-326 mg/dL

Reference values have not been established for patients who are younger than 18 years of age.

PROTEIN/CREATININE RATIO:

> or =18 years: <0.18 mg/mg creatinine

Reference values have not been established for patients younger than 18 years of age.

ELECTROPHORESIS, PROTEIN

The following fractions, if present, will be reported as mg/dL:

- -Albumin
- -Alpha-1-globulin
- -Alpha-2-globulin
- -Beta-globulin
- -Gamma-globulin

No reference values apply to random urines.

MASS-FIX M-PROTEIN ISOTYPE

M-protein Isotype MS:

No monoclonal protein detected

Flag M-protein Isotype MS:

Negative