

Hypercholesterolemia Gene Panel, Varies

# **Reporting Title:** Hypercholesterolemia Gene Panel **Performing Location:** Rochester

### Ordering Guidance:

Customization of this panel and single gene analysis for any gene present on this panel are available. For more information see CGPH / Custom Gene Panel, Hereditary, Next-Generation Sequencing, Varies.

Targeted testing for familial variants (also called site-specific or known mutations testing) is available for the genes on this panel. See FMTT / Familial Variant, Targeted Testing, Varies. To obtain more information about this testing option, call 800-533-1710.

#### Shipping Instructions:

Specimen preferred to arrive within 96 hours of collection.

#### Necessary Information:

Prior Authorization is available, but not required, for this test. If proceeding with the prior authorization process, submit the required form with the specimen.

#### **Specimen Requirements:**

Patient Preparation: A previous bone marrow transplant from an allogenic donor will interfere with testing. Call 800-533-1710 for instructions for testing patients who have received a bone marrow transplant.

Specimen Type: Whole blood Container/Tube: Preferred: Lavender top (EDTA) or yellow top (ACD) Acceptable: Any anticoagulant Specimen Volume: 3 mL Collection Instructions: 1. Invert several times to mix blood. 2. Send whole blood specimen in original tube. Do not aliquot. Specimen Stability Information: Ambient (preferred)/Refrigerated

#### Specimen Minimum Volume:

1 mL

## Forms:

1. New York Clients-Informed consent is required. Document on the request form or electronic order that a copy is on file.

The following documents are available:

-Informed Consent for Genetic Testing (T576)



Hypercholesterolemia Gene Panel, Varies

-Informed Consent for Genetic Testing (Spanish) (T826)

2. Hereditary Dyslipidemia Patient Information

3. Hypercholesterolemia Gene Panel (HCHLG) Prior Authorization Ordering Instructions

4. If not ordering electronically, complete, print, and send a Cardiovascular Test Request Form (T724) with the specimen.

Specimen Type	Temperature	emperature Time	
Varies	Varies		

# **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
617268	Test Description	Alphanumeric		62364-5
617269	Specimen	Alphanumeric		31208-2
617270	Source	Alphanumeric		31208-2
617271	Result Summary	Alphanumeric		50397-9
617272	Result	Alphanumeric		82939-0
617273	Interpretation	Alphanumeric		69047-9
617274	Additional Results	Alphanumeric		82939-0
617275	Resources	Alphanumeric		99622-3
617276	Additional Information	Alphanumeric		48767-8
617277	Method	Alphanumeric		85069-3
617278	Genes Analyzed	Alphanumeric		48018-6
617279	Disclaimer	Alphanumeric		62364-5
617280	Released By	Alphanumeric		18771-6

LOINC and CPT codes are provided by the performing laboratory.

## Supplemental Report:

Supplemental

# **CPT Code Information:**

81406 x2 81407 81479 81479 (if appropriate for government payers)



Hypercholesterolemia Gene Panel, Varies

## **Reference Values:**

An interpretive report will be provided.