

Atypical Hemolytic Uremic Syndrome (aHUS)/Thrombotic Microangiopathy (TMA) /Complement 3 Glomerulopathy (C3G) Gene Panel, Varies

Reporting Title: aHUS/TMA/C3G Gene Panel

Performing Location: Rochester

Ordering Guidance:

Due to atypical hemolytic uremic syndrome genotype-phenotype complexity, targeted testing for familial variants will not be accepted without approval from the laboratory; call 800-533-1710 to discuss testing options with a genetic counselor.

Customization of this panel and single gene analysis for any gene present on this panel are available. For more information, see CGPH / Custom Gene Panel, Hereditary, Next-Generation Sequencing, Varies

Shipping Instructions:

Specimen preferred to arrive within 96 hours of collection.

Specimen Requirements:

Patient Preparation: A previous bone marrow transplant from an allogenic donor will interfere with testing. Call 800-533-1710 for instructions for testing patients who have received a bone marrow transplant.

Submit only 1 of the following specimens:

Specimen Type: Whole blood

Container/Tube:

Preferred: Lavender top (EDTA) or yellow top (ACD)

Acceptable: Any anticoagulant Specimen Volume: 3 mL Collection Instructions:

1. Invert several times to mix blood.

2. Send whole blood specimen in original tube. Do not aliquot.

Specimen Stability Information: Ambient (preferred) 4 days/Refrigerated

Specimen Type: Skin biopsy

Supplies: Fibroblast Biopsy Transport Media (T115)

Container/Tube: Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The solution should be supplemented with 1% penicillin and streptomycin. Tubes of culture media can be supplied upon request (Eagle's minimum essential medium with 1% penicillin and streptomycin).

Specimen Volume: 4-mm punch

Specimen Stability Information: Refrigerated (preferred)/Ambient

Additional Information: A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing, Tissue. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.

Specimen Type: Cultured fibroblasts

Container/Tube: T-25 flask Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured fibroblast cells from a skin biopsy from another laboratory. Cultured

cells from a prenatal specimen will not be accepted.

Specimen Stability Information: Ambient (preferred)/Refrigerated (<24 hours)



Atypical Hemolytic Uremic Syndrome (aHUS)/Thrombotic Microangiopathy (TMA) /Complement 3 Glomerulopathy (C3G) Gene Panel, Varies

Additional Information: A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing, Tissue. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.

Specimen Minimum Volume:

Blood: 1 mL; Skin biopsy or cultured fibroblasts: See Specimen Required

Forms:

- 1. New York Clients-Informed consent is required. Document on the request form or electronic order that a copy is on file. The following documents are available:
- -Informed Consent for Genetic Testing (T576)
- -Informed Consent for Genetic Testing-Spanish (T826)
- 2. Hereditary Renal Genetic Testing Patient Information (T918)

Specimen Type	Temperature	Time	Special Container
Varies	Varies		

Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
CULFB	CG770	Reason for Referral	Plain Text	No
CULFB	CG899	Specimen	Plain Text	No

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
618017	Test Description	Alphanumeric		62364-5
618018	Specimen	Alphanumeric		31208-2
618019	Source	Alphanumeric		31208-2
618020	Result Summary	Alphanumeric		50397-9
618021	Result	Alphanumeric		82939-0
618022	Interpretation	Alphanumeric		69047-9
618023	Additional Results	Alphanumeric		82939-0
618024	Resources	Alphanumeric		99622-3



Atypical Hemolytic Uremic Syndrome (aHUS)/Thrombotic Microangiopathy (TMA) /Complement 3 Glomerulopathy (C3G) Gene Panel, Varies

Result ID	Reporting Name	Туре	Unit	LOINC®
618025	Additional Information	Alphanumeric		48767-8
618026	Method	Alphanumeric		85069-3
618027	Genes Analyzed	Alphanumeric		48018-6
618028	Disclaimer	Alphanumeric		62364-5
618029	Released By	Alphanumeric		18771-6

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

Supplemental

CPT Code Information:

81404 81479 81479 (if appropriate for government payers)

Reflex Tests:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
CULFB	Fibroblast Culture for Genetic Test			No	Yes

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
CULFB	52327	Result Summary	Alphanumeric		50397-9
CULFB	52329	Interpretation	Alphanumeric		69965-2
CULFB	52328	Result	Alphanumeric		82939-0
CULFB	CG770	Reason for Referral	Alphanumeric		42349-1
CULFB	CG899	Specimen	Alphanumeric		31208-2
CULFB	52331	Source	Alphanumeric		31208-2
CULFB	52332	Method	Alphanumeric		85069-3
CULFB	54625	Additional Information	Alphanumeric		48767-8
CULFB	52333	Released By	Alphanumeric		18771-6



Atypical Hemolytic Uremic Syndrome (aHUS)/Thrombotic Microangiopathy (TMA) /Complement 3 Glomerulopathy (C3G) Gene Panel, Varies

Reference Values:

An interpretive report will be provided.