

Hepatitis A Virus Total Antibodies, Serum

# **Reporting Title:** Hepatitis A Virus Total Ab, S **Performing Location:** Rochester

#### **Necessary Information:**

Date of collection is required.

#### Specimen Requirements:

Patient Preparation: For 24 hours before specimen collection, patient should not take multivitamins or dietary supplements (eg. hair, skin, and nail supplements) containing biotin (Vitamin B7).
Collection Container/Tube: Serum gel
Submission Container/Tube: Plastic vial
Specimen Volume: 0.6 mL
Collection Instructions:
1. Centrifuge blood collection tube per manufacturer's instructions (eg, centrifuge and aliquot within 2 hours of collection for BD Vacutainer tubes).
2. Aliquot serum into plastic vial.

#### **Specimen Minimum Volume:**

0.6 mL

#### Forms:

If not ordering electronically, complete, print, and send an Infectious Disease Serology Test Request (T916) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum SST	Frozen (preferred)	90 days	
	Refrigerated	6 days	
	Ambient	72 hours	

#### **Result Codes:**

Result ID	)	Reporting Name	Туре	Unit	LOINC®
HAVT		Hepatitis A Virus Total Ab, S	Alphanumeric		13951-9

LOINC and CPT codes are provided by the performing laboratory.

### Supplemental Report:



# **Test Definition: HAVTA**

Hepatitis A Virus Total Antibodies, Serum

No

## **CPT Code Information:**

86708

# **Reference Values:**

Unvaccinated: Negative Vaccinated: Positive