

**Reporting Title:** HE4, S**Performing Location:** Rochester**Specimen Requirements:**

Patient Preparation: For 12 hours before specimen collection do not take multivitamins or dietary supplements containing biotin (vitamin B7), which is commonly found in hair, skin, and nail supplements and multivitamins.

Supplies: Sarstedt 5 mL Aliquot Tube (T914)

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container: Plastic vial

Specimen Volume: 1 mL

Collection Instructions: Centrifuge and aliquot serum into plastic vial.

**Specimen Minimum Volume:**

0.75 mL

**Forms:**

If not ordering electronically, complete, print, and send an Oncology Test Request (T729) with the specimen.

| Specimen Type | Temperature        | Time    | Special Container |
|---------------|--------------------|---------|-------------------|
| Serum         | Frozen (preferred) | 84 days |                   |
|               | Refrigerated       | 7 days  |                   |

**Result Codes:**

| Result ID | Reporting Name | Type    | Unit   | LOINC®  |
|-----------|----------------|---------|--------|---------|
| HE4       | HE4, S         | Numeric | pmol/L | 55180-4 |

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

86305

---

**Reference Values:**

Females: < or =140 pmol/L

Males: Not applicable