

# **Reporting Title:** Methadone Confirmation, CoC, U **Performing Location:** Rochester

ABORATORIES

### **Specimen Requirements:**

Supplies: Chain of Custody Kit (T282)

Container/Tube: Chain-of-custody kit containing the specimen containers, seals, and documentation required. Specimen Volume: 10 mL

Collection Instructions: Collect urine specimen in the container provided, seal, and submit with the associated documentation to satisfy the legal requirements for chain-of-custody testing.

Additional Information: Submitting less than 10 mL will compromise our ability to perform all necessary testing.

#### **Specimen Minimum Volume:**

2.5 mL

#### Forms:

1. Chain of Custody Request is included in the Chain-of-Custody Kit (T282).

2. If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	28 days	
	Frozen	28 days	
	Ambient	14 days	

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
21105	Methadone Immunoassay Screen	Alphanumeric		70149-0
36208	EDDP-by GC-MS	Alphanumeric	ng/mL	58429-2
36209	Methadone-by GC-MS	Alphanumeric	ng/mL	16246-1
36210	Methadone Interpretation	Alphanumeric		69050-3
36211	Chain of Custody	Alphanumeric		77202-0

LOINC and CPT codes are provided by the performing laboratory.

#### Supplemental Report:

No



# **CPT Code Information:**

80358 G0480 (if appropriate)

# **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
СОСН	Chain of Custody Processing			Yes	No
ADLTX	Adulterants Survey, CoC, U			Yes	Yes

# **Result Codes for Reflex Tests:**

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
ADLTX	36121	Creatinine, U	Numeric	mg/dL	2161-8
ADLTX	36122	Specific Gravity	Numeric		5810-7
ADLTX	36123	pH	Numeric		2756-5
ADLTX	36124	Oxidants	Alphanumeric		58714-7
ADLTX	36125	Nitrites	Alphanumeric		32710-6
ADLTX	36126	Comment	Alphanumeric		48767-8

# **Reference Values:**

Negative

Cutoff concentrations: METHADONE BY GC-MS <100 ng/mL 2-ETHYLIDENE-1,5-DIMETHYL-3,3-DIPHENYLPYRROLIDINE GC-MS <100 ng/mL