

**Reporting Title:** Fentanyl Screen, U**Performing Location:** Rochester**Ordering Guidance:**

For situations where chain of custody is required, a Chain of Custody Kit (T282) is available. For chain-of-custody testing, order FENTX / Fentanyl with Metabolite Confirmation, Chain of Custody, Urine.

For monitoring therapeutic drug levels, order FENTS / Fentanyl, Serum.

**Additional Testing Requirements:**

If urine creatinine is required or adulteration of the sample is suspected, the following test should also be ordered, ADULT / Adulterants Survey, Random, Urine.

**Specimen Requirements:**

Supplies: Urine Tubes, 10 mL (T068)

Collection Container Tube: Plastic urine container

Submission Container/Tube: 10 mL tube

Specimen Volume: 2 mL

Collection Instructions:

1. Collect a random urine specimen.
2. No preservative.

**Specimen Minimum Volume:**

0.5 mL

**Forms:**

If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	14 days	
	Ambient	72 hours	

---

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
63060	Fentanyl Screen, U	Alphanumeric	ng/mL	59673-4

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

80307

**Reference Values:**

Negative

Screening cutoff concentration: 2 ng/mL