

Reporting Title: Ethyl Glucuronide Screen, U**Performing Location:** Rochester**Ordering Guidance:**

For situations where chain of custody is required, a Chain of Custody Kit (T282) is available. For chain-of-custody testing, order ETGX / Ethyl Glucuronide Confirmation, Chain of Custody, Random, Urine.

Additional Testing Requirements:

If urine creatinine is required or adulteration of the sample is suspected, the following test should also be ordered, ADULT / Adulterants Survey, Random, Urine.

Specimen Requirements:

Supplies: Sarstedt 5 mL Aliquot Tube (T914)

Collection Container/Tube: Plastic urine container

Submission Container/Tube: Plastic, 5-mL tube

Specimen Volume: 2 mL

Collection Instructions:

1. Collect a random urine specimen.
2. No preservative.

Specimen Minimum Volume:

0.5 mL

Forms:

If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	28 days	
	Frozen	28 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
63420	Ethyl Glucuronide Screen, U	Alphanumeric	ng/ml	58375-7

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80307

Reference Values:

Negative

Screening cutoff concentration: 500 ng/mL