

Reporting Title: Hydroxychloroquine, S**Performing Location:** Rochester**Specimen Requirements:**

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube: Red top (gel tubes/SST are not acceptable)

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Collect specimen immediately before next scheduled dose.
2. Within 2 hours of collection, centrifuge, and aliquot serum into a plastic vial.

Specimen Minimum Volume:

0.5 mL

Forms:

If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	28 days	
	Frozen	28 days	
	Ambient	21 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
64947	Hydroxychloroquine, S	Numeric	ng/mL	3684-8

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80220

Reference Values:

For suppressive treatment of malaria, suggested plasma or serum concentrations should be >10 ng/mL.

For systemic lupus erythematosus, proposed serum target concentrations should be > or =500 ng/mL.