
Reporting Title: 3-Methoxytyramine, 24h, U**Performing Location:** Rochester**Necessary Information:**

24-Hour volume (in milliliters) is required.

Specimen Requirements:

Patient Preparation: Tricyclic antidepressants, labetalol, and sotalol medications may elevate levels of catecholamines producing results that cannot be interpreted. If clinically feasible, it is optimal to discontinue these medications at least 1 week before collection. Levodopa (Sinemet) medication will cause false-positive results. For advice on assessing the risk of removing patients from these medications and alternatives, consider consultation with a specialist in endocrinology or hypertension.

Supplies: Urine Tubes, 10 mL (T068)

Submission Container/Tube: Plastic urine tube

Specimen Volume: 10 mL

Collection Instructions:

1. Complete 24-hour urine collections are preferred, especially for patients with episodic hypertension; ideally the collection should begin at the onset of a "spell."

2. Add 10 g (pediatric: 3 g) of boric acid or 25 mL (pediatric: 15 mL) of 50% acetic acid as preservative at start of collection.

3. Collect urine for 24 hours.

Additional Information: See Urine Preservatives-Collection and Transportation for 24-Hour Urine Specimens for multiple collections.

Specimen Minimum Volume:

3 mL

Urine Preservative Collection Options:

Note: The addition of preservative must occur prior to beginning the collection.

Ambient (no additive) OK

Refrigerate (no additive) OK

Frozen (no additive) OK

50% Acetic Acid Preferred

Boric Acid Preferred

Diazolidinyl Urea No

6M Hydrochloric Acid OK

6M Nitric Acid OK

Sodium Carbonate OK

Thymol No

Toluene OK

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	28 days	
	Frozen	90 days	
	Ambient	28 days	

Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Type	Reportable
3MT	TM120	Collection Duration	Plain Text	Yes
3MT	VL120	Urine Volume	Plain Text	Yes

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
65157	3-Methoxytyramine, U	Alphanumeric	mcg/24 h	32618-1
TM120	Collection Duration	Alphanumeric		13362-9
VL120	Urine Volume	Alphanumeric		3167-4

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

82542

Reference Values:

Males: < or =306 mcg/24 hours

Females: < or =242 mcg/24 hours