

# **Test Definition: BUPMX**

Buprenorphine and Norbuprenorphine, Chain of Custody, Random, Urine

Reporting Title: Buprenorphine w/metabolite, CoC, U

Performing Location: Rochester

## **Specimen Requirements:**

Supplies: Chain of Custody Kit (T282)

Container/Tube: Chain of Custody Kit containing the specimen containers, seals, and documentation required.

Specimen Volume: 5 mL

Collection Instructions: Collect specimen in the provided container, seal, and submit with the associated documentation

to satisfy the legal requirements for chain-of-custody testing.

## **Specimen Minimum Volume:**

1 mL

#### Forms:

1. Chain of Custody Request is included in the Chain-of-Custody Kit (T282).

2. If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Ambient	14 days	
	Frozen	14 days	

#### Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
56644	Buprenorphine Immunoassay Screen	Alphanumeric	ng/mL	58359-1
65215	Buprenorphine	Alphanumeric	ng/mL	3415-7
48297	Norbuprenorphine	Alphanumeric	ng/mL	49753-7
56645	Chain of Custody	Alphanumeric		77202-0

LOINC and CPT codes are provided by the performing laboratory.

## Supplemental Report:

No

#### **CPT Code Information:**



## **Test Definition: BUPMX**

Buprenorphine and Norbuprenorphine, Chain of Custody, Random, Urine

80348 G0480 (if appropriate)

#### **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
СОСН	Chain of Custody Processing			Yes	No
ADLTX	Adulterants Survey, CoC, U			Yes	Yes

### **Result Codes for Reflex Tests:**

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
ADLTX	36121	Creatinine, U	Numeric	mg/dL	2161-8
ADLTX	36122	Specific Gravity	Numeric		5810-7
ADLTX	36123	рН	Numeric		2756-5
ADLTX	36124	Oxidants	Alphanumeric		58714-7
ADLTX	36125	Nitrites	Alphanumeric		32710-6
ADLTX	36126	Comment	Alphanumeric		48767-8

## **Reference Values:**

Negative

Cutoff concentrations: Buprenorphine: 5.0 ng/mL Norbuprenorphine: 2.5 ng/mL