

Test Definition: TRPS

Troponin T, 5th Generation, Plasma

Reporting Title: Troponin T, 5th gen, P

Performing Location: Rochester

Specimen Requirements:

Collection Container/Tube:

Preferred: Light-green top (lithium heparin gel)

Acceptable: Green top (lithium heparin) Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL Collection Instructions:

1. Lithium heparin gel tubes should be centrifuged within 2 hours of collection.

2. Plasma from lithium heparin tubes should be centrifuged and aliquoted into a plastic vial within 2 hours of collection.

Specimen Minimum Volume:

0.5 mL

Forms:

If not ordering electronically, complete, print, and send a Cardiovascular Test Request Form (T724) with the specimen.

Specimen Type	Temperature	Time	Special Container	
Plasma Li Heparin	Frozen (preferred)	365 days		
	Ambient	24 hours		
	Refrigerated	24 hours		

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
TRPS	Troponin T, 5th gen, P	Numeric	ng/L	67151-1

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

84484



Test Definition: TRPS

Troponin T, 5th Generation, Plasma

Reference Values:

Males: < or =15 ng/L Females: < or =10 ng/L