

---

**Reporting Title:** Buprenorphine and Metabolite, U**Performing Location:** Rochester**Ordering Guidance:**

For screening buprenorphine alone, order BUPS / Buprenorphine Screen, Random, Urine.

For comprehensive opioid screening, order TOSU / Targeted Opioid Screen, Random, Urine.

For situations where chain of custody is required, a Chain of Custody Kit (T282) is available. For chain-of-custody testing, order BUPMX / Buprenorphine and Norbuprenorphine, Chain of Custody, Random, Urine.

Additional drug panels and specific requests are available. Call 800-533-1710 or 507-266-5700.

**Additional Testing Requirements:**

If urine creatinine is required or adulteration of the sample is suspected, the following test should also be ordered, ADULT / Adulterants Survey, Random, Urine.

**Specimen Requirements:**

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube: Plastic urine container

Submission Container/Tube: Plastic, 5-mL tube

Specimen Volume: 1 mL

Collection Instructions:

1. Collect a random urine specimen.
2. No preservative

Additional Information:

1. No specimen substitutions.
2. STAT requests are not accepted for this test.
3. Submitting less than 1 mL will compromise the ability to perform all necessary testing.

**Specimen Minimum Volume:**

0.5 mL

**Forms:**

If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	

---

	Ambient	14 days	
	Frozen	14 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
66200	Buprenorphine	Alphanumeric	ng/mL	3415-7
48296	Norbuprenorphine	Alphanumeric	ng/mL	49753-7

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**  
No**CPT Code Information:**

80348  
G0480 (if appropriate)

**Reference Values:**

Negative

Cutoff concentrations:  
Buprenorphine: 5.0 ng/mL  
Norbuprenorphine: 2.5 ng/mL