

## **Test Definition: P40NA**

p40 + Napsin A Immunostain, Technical Component Only

Reporting Title: p40 + Napsin A IHC, Tech Only

Performing Location: Rochester

## Ordering Guidance:

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

## **Shipping Instructions:**

Attach the green "Attention Pathology" address label (T498) and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

## **Specimen Requirements:**

Supplies: Immunostain Technical Only Envelope (T693)

Specimen Type: Tissue

Container/Tube: Immunostains Technical Only Envelope

Preferred: 2 Unstained positively charged glass slide (25- x 75- x 1-mm) per test ordered; sections 4-microns thick.

Acceptable: Formalin-fixed, paraffin-embedded tissue block

#### Forms:

If not ordering electronically, complete, print, and send a Immunohistochemical (IHC)/In Situ Hybridization (ISH) Stains Request (T763) with the specimen.

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

#### Result Codes:

Result ID		Reporting Name	Туре	Unit	LOINC®	
	70838	p40 + Napsin A IHC, Tech Only	Alphanumeric		Bill only; no re	

LOINC and CPT codes are provided by the performing laboratory.

## Supplemental Report:

No

#### **CPT Code Information:**



# **Test Definition: P40NA**

p40 + Napsin A Immunostain, Technical Component Only

88344-TC

## **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
IHMTO	IHC Multiplex, Tech Only			No	No