

## Test Definition: TDT

Terminal Deoxynucleotidyl Transferase (TdT) Immunostain, Technical Component Only

# **Reporting Title:** TdT IHC, Tech Only **Performing Location:** Rochester

#### Ordering Guidance:

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

#### **Shipping Instructions:**

Attach the green "Attention Pathology" address label (T498) and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

#### **Specimen Requirements:**

Supplies: Immunostain Technical Only Envelope (T693) Specimen Type: Tissue Container/Tube: Immunostains Technical Only Envelope Preferred: 2 Unstained positively charged glass slide (25- x 75- x 1-mm) per test ordered; sections 4-microns thick. Acceptable: Formalin-fixed, paraffin-embedded tissue block

### Forms:

If not ordering electronically, complete, print, and send a Immunohistochemical (IHC)/In Situ Hybridization (ISH) Stains Request (T763) with the specimen.

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
70875	TdT IHC, Tech Only	Alphanumeric		Bill only; no re

LOINC and CPT codes are provided by the performing laboratory.

#### Supplemental Report:

No

#### **CPT Code Information:**



### **Test Definition: TDT**

Terminal Deoxynucleotidyl Transferase (TdT) Immunostain, Technical Component Only

88342-TC, Primary 88341-TC, If additional IHC

#### **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
IHTOI	IHC Initial, Tech Only			No	No
ΙΗΤΟΑ	IHC Additional, Tech Only			No	No