

**Reporting Title:** NuSwab Vaginitis (VG)**Performing Location:** LabCorp Burlington**Specimen Requirements:**

Collection Container/Tube: APTIMA Multitest, vaginal, or unisex swab

Specimen Volume: One swab

Collection Instructions: Collect vaginal fluid sample using the Gen-Probe Aptima swab by contacting the swab to the lower third of the vaginal wall and rotating the swab for 10 to 30 seconds to absorb fluid. Immediately place the swab into the transport tube and carefully break the swab shaft against the side of the tube. Tightly screw on the cap. Submit one vaginal swab in APTIMA Multitest, vaginal, or unisex swab. Ship refrigerate.

**Specimen Minimum Volume:**

One swab

Specimen Type	Temperature	Time	Special Container
Swab	Refrigerated (preferred)	30 days	
	Ambient	30 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
Z4735	Atopobium vaginae	Alphanumeric		69565-0
Z4736	BVAB 2	Alphanumeric		69566-8
Z4737	Megasphaera 1	Alphanumeric		69567-6
Z4738	Candida albicans, NAA	Alphanumeric		69562-7
Z4739	Candida glabrata, NAA	Alphanumeric		69563-5
Z4740	Trich vag by NAA	Alphanumeric		62461-9

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

87801

87798 x 3

87661

**Reference Values:**

Candida albicans, NAA: Negative  
Candida glabrata, NAA: Negative  
Trich vag by NAA: Negative