

**Reporting Title:** Prostaglandin D2 (PG D2), U**Performing Location:** Inter Science Institute**Specimen Requirements:**

Patient Preparation: Patient should not be on aspirin, indomethacin, or anti-inflammatory medications, if possible, for at least 48 hours prior to collection of specimen. Specimen must be frozen within 30 minutes of collection.

Specimen Type: Urine

Submission Container/Tube: Plastic, 10-mL tube (T068)

Specimen Volume: 10 mL

Collection Instructions:

1. Collect random urine (NO preservative).
3. Freeze immediately and send specimen frozen in the plastic, 10-mL urine tube (T068)

Note: 24 hours urine collection is not acceptable.

**Specimen Minimum Volume:**

5 mL

Specimen Type	Temperature	Time	Special Container
Urine	Frozen	180 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
FPD2U	Prostaglandin D2 (PG D2), U	Alphanumeric	ng/g Creatinine	Not Provided

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

84150

**Reference Values:**

Up to 175 ng/g Creatinine