

Reporting Title: Digitoxin**Performing Location:** NMS Labs**Specimen Requirements:**

Must submit one specimen per order. Specimens cannot be shared between multiple orders.

Specimen Type: Serum

Collection Container/Tube: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Draw blood in a plain, red-top tube(s). Serum gel tube is not acceptable.
2. Centrifuge and send 1 mL of serum refrigerated in a plastic, preservative-free vial.

Specimen Minimum Volume:

0.4 mL

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	90 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
FDGTX	Digitoxin	Alphanumeric		3559-2

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

Referral

CPT Code Information:

80299

Reference Values:

Reporting limit determined each analysis.

None Detected ng/mL

Therapeutic Range: 10 - 30 ng/mL