

Melatonin, Plasma

## **Reporting Title:** Melatonin, Plasma **Performing Location:** NMS Labs

#### **Specimen Requirements:**

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*
Specimen Type: Plasma
Collection Container/Tube:
Preferred: Lavender top (EDTA)
Acceptable: Pink top (EDTA)
Submission Container/Tube: Plastic vial
Specimen volume: 3 mL
Collection instructions:
1. Draw blood in EDTA (lavender top) tube(s).
2. Centrifuge and send 1 mL of plasma refrigerated in a plastic vial.

### **Specimen Minimum Volume:**

0.5 mL

Specimen Type	Temperature	Time	Special Container
Plasma EDTA	Refrigerated (preferred)	30 days	
	Frozen	365 days	
	Ambient	14 days	

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z3521	Melatonin	Alphanumeric		11055-1

LOINC and CPT codes are provided by the performing laboratory.

# Supplemental Report:

No

### **CPT Code Information:**

80299



Melatonin, Plasma

### **Reference Values:**

Reporting limit determined each analysis.

Units: ng/mL