

# **Test Definition: FACES**

Acetoacetate, Serum

Reporting Title: Acetoacetate, Serum

Performing Location: NMS Labs

# **Specimen Requirements:**

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

Specimen Type: Serum

Collection Container/Tube: Red

Submission Container/Tube: Plastic vial

Specimen Volume: 3 mL Collection Instructions:

1. Draw blood in a plain, red-top tube(s). Serum gel tube is not acceptable.

2. Centrifuge and send 3 mL of serum frozen in a plastic, preservative-free vial.

# **Specimen Minimum Volume:**

1.2 mL

Specimen Type	Temperature	Time	Special Container
Serum Red	Frozen (preferred)	4 days	
	Refrigerated	4 days	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z3524	Acetoacetate	Alphanumeric		Not Provided

LOINC and CPT codes are provided by the performing laboratory.

#### **Supplemental Report:**

No

#### **CPT Code Information:**

82010



# **Test Definition: FACES**

Acetoacetate, Serum

### **Reference Values:**

Reporting limit determined each analysis.

Acetoacetate: None Detected mcg/mL -Normal range for adults: 5-30 mcg/mL