

# **Test Definition: FCLMS**

Clonazepam and Metabolite, Serum

Reporting Title: Clonazepam and Metabolite, S

Performing Location: NMS Labs

### **Specimen Requirements:**

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

Specimen Type: Serum

Collection Container/Tube: Red top

Submission Container/Tube: Plastic, preservative-free vial

Specimen Volume: 2 mL Collection Instructions:

1. Draw blood in a red top tube(s). Serum gel tube is not acceptable.

2. Centrifuge and send 2 mL of serum refrigerated in a plastic, preservative-free vial.

#### Specimen Minimum Volume:

0.7 mL

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Frozen	30 days	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z6336	Clonazepam	Alphanumeric		3494-2
Z6337	7-Amino Clonazepam	Alphanumeric		28059-4

LOINC and CPT codes are provided by the performing laboratory.

## Supplemental Report:

No

#### **CPT Code Information:**

80346

G0480 (if appropriate)



# **Test Definition: FCLMS**

Clonazepam and Metabolite, Serum

#### **Reference Values:**

Reporting limit(s) determined each analysis. Clonazepam: None Detected ng/mL 7-Amino Clonazepam: None Detected ng/mL