

**Reporting Title:** Isavuconazole**Performing Location:** Eurofins Viracor**Specimen Requirements:**

Specimen Type: Serum

Container/Tube: Red

Specimen Volume: 1 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is not acceptable. Allow to clot for 30 to 60 minutes, spin down and send 1 mL of serum Frozen in a sterile, screw top tube.

**Specimen Minimum Volume:**

0.5 mL

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	30 days	
	Ambient	14 days	
	Refrigerated	14 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
FIVCZ	Isavuconazole	Alphanumeric		Not Provided

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

80299

---

**Reference Values:**

The analytical measuring range is 0.1-10 mcg/mL. The therapeutic range has not yet been established.