

Reporting Title: Anti-Mi-2 Ab**Performing Location:** Esoterix Endocrinology**Specimen Requirements:**

Collection Container/Tube: 6 mL Red

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Acceptable: SST

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Separate serum from cells within 1 hour of collection and send 1 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume:

0.3 mL (volume does NOT allow for repeat testing)

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Frozen	60 days	
	Ambient	7 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
FFMI2	Anti-Mi-2 Ab	Alphanumeric		18485-3

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83516

Reference Values:

Negative