

Reporting Title: Anti-Synthetase Profile**Performing Location:** Esoterix Endocrinology**Specimen Requirements:**

Collection Container/Tube: 10 mL Red

Submission Container/Tube: Plastic vial

Specimen Volume: 3 mL

Acceptable: SST

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 3 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume:

1 mL (volume does NOT allow for repeat testing)

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Frozen	60 days	
	Ambient	7 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
Z5782	Anti-Jo-1 Ab	Alphanumeric		35333-4
Z5783	Anti-PL-7 Ab	Alphanumeric		33772-5
Z5837	Anti-PL-12 Ab	Alphanumeric		33771-7
Z5823	Anti-EJ Ab	Alphanumeric		45149-2
Z5824	Anti-OJ Ab	Alphanumeric		45152-6

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83516 x 4

86235

Reference Values:

Anti-PL-7 Ab, Anti-PL-12 Ab, Anti-EJ Ab, Anti-OJ Ab:
Reference Range: Negative

Interpretation for:
Anti-Jo-1 Ab:
Reference Range: <20
Negative: <20 units
Weak Positive: 20-39 units
Moderate Positive: 40-80 units
Strong Positive: >80 units