

## **Test Definition: FAGES**

Androstanediol Glucuronide (Endocrine Sciences)

Reporting Title: Androstanediol Gluc, Endo Sci

Performing Location: Esoterix Endocrinology

### **Specimen Requirements:**

Collection container/tube: 5 mL Red Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Acceptable: SST

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 1 mL of

serum refrigerated in a plastic vial.

#### **Specimen Minimum Volume:**

0.5 mL (Note: Minimum volume does not allow for repeat analysis)

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	6 days	
	Ambient	6 days	
	Frozen		

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z5864	3A Androstanediol-G	Alphanumeric		1680-8

LOINC and CPT codes are provided by the performing laboratory.

#### **Supplemental Report:**

No

#### **CPT Code Information:**

82154



# **Test Definition: FAGES**

Androstanediol Glucuronide (Endocrine Sciences)

#### **Reference Values:**

Age	Range
Prepubertal Children	Not Established
Adult Males	112-1046 ng/dL
Adult Females	11-249 ng/dL

Occasionally, normal females with no evidence of hirsutism may have levels well beyond the normal range.