

## **Test Definition: F11DX**

11-Desoxycortisol

Reporting Title: 11-Desoxycortisol
Performing Location: Esoterix Endocrinology

#### **Specimen Requirements:**

Collection Container/Tube: Red-Top Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Acceptable: SST

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Serum must be separated

from cells within 45 minutes of venipuncture. Spin down and send 1 mL of serum frozen in a plastic vial.

#### **Specimen Minimum Volume:**

0.2 mL (Note: Minimum volume does not allow for repeat analysis)

Specimen Type	Temperature	Temperature Time	
Serum	Frozen (preferred)		
	Ambient	72 hours	
	Refrigerated	72 hours	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
F11DX	11-Desoxycortisol	Alphanumeric		1657-6

LOINC and CPT codes are provided by the performing laboratory.

#### **Supplemental Report:**

No

#### **CPT Code Information:**

82634



# **Test Definition: F11DX**

11-Desoxycortisol

### **Reference Values:**

Age	Range (ng/dL)
Premature (26 - 28 weeks) Day 4	110-1376
Premature (31 - 35 weeks) Day 4	48-579
Newborn Day 3	13-147
Newborn 1 to 11 months	<10-156
Prepubertal 8 AM	20-155
Prepubertal Children and Adults 8 AM	12-158