

**Reporting Title:** Pancreastatin, Plasma**Performing Location:** Inter Science Institute**Specimen Requirements:**

Patient Preparation:

1. Patient should not be on any medications that may influence Insulin levels, if possible, for at least 48 hours prior to collection.
2. Patient should be fasting 10 hours prior to collection. Patient may drink plain water; no other liquid is acceptable.

Specimen Type: Plasma

Collection Container/Tube: Z tube (MCL T701)

Submission Container/Tube: Plastic vial

Specimen Volume: 3 mL

Collection Instructions:

1. Draw 10 mL of blood in special Z-tube, pre-chilled (MCL T701).
2. Separate plasma from cells immediately after draw and freeze immediately after separation.
3. Send 3 mL of plasma frozen in a plastic vial.

**Specimen Minimum Volume:**

1 mL

Specimen Type	Temperature	Time	Special Container
Z-Tube Plasma	Frozen	60 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
FPANP	Pancreastatin, Plasma	Numeric	pg/mL	49013-6

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

83519

---

**Reference Values:**

10-135 pg/mL