

Reporting Title: Fluconazole (FLU) Susceptibility
Performing Location: U.T. Health Science Ctr San Antonio

Specimen Requirements:

Container/Tube: Agar slant or other appropriate media
Specimen Volume: Organism in pure culture

Complete and submit with specimen: Fungus Testing Lab request form

Collection Instructions:

1. Organism must be in pure culture, actively growing.
2. Place specimen in a large infectious container (T146) and label as an etiologic agent/infectious substance.

Specimen Type	Temperature	Time	Special Container
Varies	Ambient		

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
FLUCS	Fluconazole (FLU) Susceptibility	Alphanumeric		Not Provided

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

87999 - Unlisted Microbiology Procedure (Refer to patient report to apply the appropriate CPT code in place of this unlisted CPT code based on specimen source.)

87188 - mould - MIC microdilution or agar dilution (if appropriate)

87186 - yeast - MIC microdilution or agar dilution (if appropriate)