

**Reporting Title:** Haloperidol, S**Performing Location:** Rochester**Specimen Requirements:**

Supplies: Sarstedt Aliquot Tube 5 mL (T914)

Collection Container/Tube: Red top (serum gel/SST are not acceptable)

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Draw blood immediately before next scheduled dose.

2. Within 2 hours of collection, centrifuge and aliquot serum into a plastic vial.

**Specimen Minimum Volume:**

0.3 mL

**Forms:**

If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
80339	Haloperidol, S	Numeric	ng/mL	3669-9
169	Reduced Haloperidol	Numeric	ng/mL	38364-6

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

80173

**Reference Values:**

Haloperidol:

5-17 ng/mL

Reduced Haloperidol:

10-80 ng/mL