

**Reporting Title:** Fructosamine, S**Performing Location:** Rochester**Specimen Requirements:**

Supplies: Sarstedt 5 mL Aliquot Tube (T914)

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Serum gel tubes should be centrifuged within 2 hours of collection.
2. Red-top tubes should be centrifuged and the serum aliquoted into a plastic vial within 2 hours of collection.

**Specimen Minimum Volume:**

0.5 mL

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	60 days	
	Ambient	72 hours	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
FRUCT	Fructosamine, S	Numeric	mcmol/L	15069-8

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

82985

---

**Reference Values:**

200-285 mcmol/L