

**Test Definition: COM** 

Complement, Total, Serum

Reporting Title: Complement, Total, S

Performing Location: Rochester

### **Specimen Requirements:**

Patient Preparation: Fasting preferred. Supplies: Sarstedt 5 mL Aliquot Tube (T914)

Collection Container/Tube: Red top (serum gel/SST are not acceptable)

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL Collection Instructions:

- 1. Immediately after specimen collection, place the tube on wet ice.
- 2. Centrifuge and aliquot serum into plastic vial.
- 3. Immediately freeze specimen.

## **Specimen Minimum Volume:**

0.5 mL

Specimen Type	Temperature	Time	Special Container
Serum Red	Frozen	28 days	

#### Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
СОМ	Complement, Total, S	Numeric	U/mL	4532-8

LOINC and CPT codes are provided by the performing laboratory.

### **Supplemental Report:**

No

#### **CPT Code Information:**

86162



# **Test Definition: COM**

Complement, Total, Serum

### **Reference Values:**

30-75 U/mL