

**Reporting Title:** Thyroperoxidase Ab, S**Performing Location:** Rochester**Specimen Requirements:**

Patient Preparation: For 12 hours before specimen collection do not take multivitamins or dietary supplements containing biotin (vitamin B7), which is commonly found in hair, skin, and nail supplements and multivitamins.

Supplies: Sarstedt 5 mL Aliquot Tube (T914)

Collection Container/Tube: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.6 mL

Collection Instructions: Centrifuge and aliquot serum into plastic vial within 2 hours of collection.

**Specimen Minimum Volume:**

0.5 mL

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	30 days	
	Ambient	7 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
TPO	Thyroperoxidase Ab, S	Numeric	IU/mL	8099-4

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

86376

---

**Reference Values:**

<9.0 IU/mL

Reference values apply to all ages.