

Reporting Title: Sulfamethoxazole, S**Performing Location:** Rochester**Specimen Requirements:**

Collection Container/Tube: Red top (gel tubes/SST are not acceptable)

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Serum for a peak level should be collected 60 minutes after dose.
2. Centrifuge and aliquot serum into a plastic vial within 2 hours of collection.

Specimen Minimum Volume:

0.2 mL

Forms:

If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
8238	Sulfamethoxazole, S	Numeric	mcg/mL	10342-4

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80299

Reference Values:

>50 mcg/mL (Peak)