

**Reporting Title:** Supplemental Newborn Screen, BS**Performing Location:** Rochester**Additional Testing Requirements:**

A repeat specimen is required within 1 week of birth for infants tested before they are 12 hours old.

**Specimen Requirements:**

Patient must be older than 12 hours and less than 1 week of age.

Supplies: Card-Blood Spot Collection Filter Paper (T493)

Preferred: Blood Spot Collection Card

Acceptable: Whatman Protein Saver 903 Paper, Munktel, PerkinElmer 226 filter paper, or local newborn screening card

Specimen Volume: 3 Blood spots

Collection Instructions:

1. Do not use device or capillary tube containing EDTA to collect specimen.
2. Completely fill at least 3 circles on the filter paper card (approximately 100 microliters blood per circle).
3. Let blood dry on the Blood Spot Collection Card at ambient temperature in a horizontal position for 3 hours.
4. Do not expose specimen to heat or direct sunlight.
5. Do not stack wet specimens.
6. Keep specimen dry.

Additional Information:

1. For collection instructions, see Blood Spot Collection Instructions.
2. For collection instructions in Spanish, see Blood Spot Collection Card-Spanish Instructions (T777).
3. For collection instructions in Chinese, see Blood Spot Collection Card-Chinese Instructions (T800).

**Specimen Minimum Volume:**

1 Blood spot

**Forms:**

If not ordering electronically, complete, print, and send a Biochemical Genetics Test Request (T798) with the specimen.

Specimen Type	Temperature	Time	Special Container
Whole blood	Ambient (preferred)		FILTER PAPER
	Frozen		FILTER PAPER
	Refrigerated		FILTER PAPER

---

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
82594	Supplemental Newborn Screen Result	Alphanumeric		54089-8
23727	Reviewed By	Alphanumeric		18771-6

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

83789

**Reference Values:**

An interpretive report will be provided.