

Protein S Antigen, Plasma

# Reporting Title: Protein S Ag, P

Performing Location: Rochester

# Specimen Requirements:

Specimen Type: Platelet-poor plasma

Patient Preparation: Patient must not be receiving heparin or Coumadin. If the patient is being treated with Coumadin, this should be noted. Coumadin will lower protein S.

Collection Container/Tube: Light-blue top (3.2% sodium citrate)

Submission Container/Tube: Plastic vials

Specimen Volume: 1 mL in 2 plastic vials each containing 0.5 mL

Collection Instructions:

1. For complete instructions, see Coagulation Guidelines for Specimen Handling and Processing.

2. Centrifuge, transfer all plasma into a plastic vial, and centrifuge plasma again.

3. Aliquot 0.5 mL of plasma into 2 plastic vials, leaving 0.25 mL in the bottom of centrifuged vial.

4. Freeze plasma immediately (no longer than 4 hours after collection) at -20 degrees C or, ideally -40 degrees C or below.

5. Send specimens in the same shipping container.

Additional Information: A double-centrifuged specimen is critical for accurate results as platelet contamination may cause spurious results.

## Specimen Minimum Volume:

0.5 mL

## Forms:

If not ordering electronically, complete, print, and send a Coagulation Test Request (T753) with the specimen.

Specimen Type	Temperature	Time	Special Container
Plasma Na Cit	Frozen	14 days	

## Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
PSF	Protein S Ag, Free, P	Numeric	%	27821-8
	Also used by tests: PSF			

LOINC and CPT codes are provided by the performing laboratory.



Protein S Antigen, Plasma

## Supplemental Report:

No

#### Components:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
PSF	Protein S Ag, Free, P			Yes	No

# **CPT Code Information:**

85306-Free 85305-Total (if appropriate)

#### **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
PST	Protein S Ag, Total, P			No	No

## **Result Codes for Reflex Tests:**

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
PST	PST	Protein S Ag, Total, P	Numeric	%	27823-4

#### **Reference Values:**

TOTAL Males: 80-160% Females <50 years: 70-160% > or =50 years: 80-160%

FREE Males: 65-160% Females <50 years: 50-160% > or =50 years: 65-160%

Normal, full-term newborn infants or healthy premature infants may have decreased levels of total protein S (15-50%); but because of low levels of C4b-binding protein, free protein S may be normal or near the normal adult level (> or



Protein S Antigen, Plasma

=50%). Total protein S reaches adult levels by 90 to 180 days postnatal.\*

\*See Pediatric Hemostasis References section in Coagulation Guidelines for Specimen Handling and Processing