

# **Reporting Title:** Cholesterol, Total, S **Performing Location:** Rochester

#### **Specimen Requirements:**

Collection Container/Tube: Preferred: Serum gel Acceptable: Red top Submission Container/Tube: Plastic vial Specimen Volume: 0.5 mL Collection Instructions: 1. Serum gel tubes should be centrifuged within 2 hours of collection. 2. Red-top tubes should be centrifuged and the serum aliquoted into a plastic vial within 2 hours of collection.

### **Specimen Minimum Volume:**

0.25 mL

### Forms:

If not ordering electronically, complete, print, and send a Cardiovascular Test Request (T724) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	90 days	

# **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
CHOL	Cholesterol, Total, S	Numeric	mg/dL	2093-3

LOINC and CPT codes are provided by the performing laboratory.

### Supplemental Report:

No

# **CPT Code Information:**

82465



#### **Reference Values:**

The National Lipid Association and the National Cholesterol Education Program have set the following guidelines for lipids in a context of cardiovascular disease for adults 18 years old and older:

TOTAL CHOLESTEROL Desirable: <200 mg/dL Borderline High: 200-239 mg/dL High: > or =240 mg/dL

The Expert Panel on Integrated Guidelines for Cardiovascular Health and Risk Reduction in Children and Adolescents has set the following guidelines for lipids in a context of cardiovascular disease for children 2 to 17 years of age:

TOTAL CHOLESTEROL Acceptable: <170 mg/dL Borderline High: 170-199 mg/dL High: > or =200 mg/dL

Reference values have not been established for patients who younger than 24 months of age.