

Reporting Title: Lipase, S**Performing Location:** Rochester**Specimen Requirements:**

Patient Preparation: Patients should be fasting before the specimen is collected.

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.5 mL

Collection Instructions:

1. Serum gel tube must be centrifuged within 2 hours of collection.
2. Red-top tube must be centrifuged, and the serum aliquoted into a plastic vial within 2 hours of collection.

Specimen Minimum Volume:

0.25 mL

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	365 days	
	Ambient	7 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
LPS	Lipase, S	Numeric	U/L	3040-3

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83690

Reference Values:

13-60 U/L