

Test Definition: BGA

Beta-Galactosidase, Leukocytes

Reporting Title: Beta-Galactosidase, Leukocytes

Performing Location: Rochester

Shipping Instructions:

For optimal isolation of leukocytes, it is recommended the specimen arrive refrigerated within 6 days of collection to be stabilized. Collect specimen Monday through Thursday only and not the day before a holiday. Specimen should be collected and packaged as close to shipping time as possible.

Necessary Information:

Provide a reason for referral with each specimen.

Specimen Requirements:

Container/Tube:

Preferred: Yellow top (ACD solution B) Acceptable: Yellow top (ACD solution A)

Specimen Volume: 6 mL

Collection Instructions: Send specimen in original tube. Do not aliquot.

Specimen Minimum Volume:

5 mL

Forms:

- 1. New York Clients-Informed consent is required. Document on the request form or electronic order that a copy is on file. The following documents are available:
- -Informed Consent for Genetic Testing (T576)
- -Informed Consent for Genetic Testing-Spanish (T826)
- 2. Biochemical Genetics Patient Information (T602)
- 3. If not ordering electronically, complete, print, and send a Biochemical Genetics Test Request (T798) with the specimen.

Specimen Type	Temperature	Time	Special Container
Whole Blood ACD	Refrigerated (preferred)	6 days	YELLOW TOP/ACD
	Ambient	6 days	YELLOW TOP/ACD



Test Definition: BGA

Beta-Galactosidase, Leukocytes

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
8486	Beta-Galactosidase, Leukocytes	Numeric	nmol/min/mg	24061-4
34979	Interpretation (BGA)	Alphanumeric		59462-2
34907	Reviewed By	Alphanumeric		18771-6

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

82657

Reference Values:

> or =1.56 nmol/min/mg