

# **Reporting Title:** Thallium, 24 Hr, U **Performing Location:** Rochester

### Necessary Information:

24-Hour volume (in milliliters) is required.

#### Specimen Requirements:

Patient Preparation: High concentrations of gadolinium and iodine are known to interfere with most metal tests. If either gadolinium- or iodine-containing contrast media has been administered, a specimen should not be collected for 96 hours.

Supplies: Urine Tubes, 10 mL (T068)

Collection Container/Tube: Clean, plastic urine container with no metal cap or glued insert

Submission Container/Tube: Plastic, 10-mL urine tube or a clean, plastic aliquot container with no metal cap or glued insert

Specimen Volume: 10 mL

Collection Instructions:

1. Collect urine for 24 hours.

2. Refrigerate specimen within 4 hours of completion of 24-hour collection.

3. Aliquot 10 mL into a plastic 10-mL urine tube (T068) or a clean, plastic aliquot container with no metal cap or glued insert.

4. See Metals Analysis Specimen Collection and Transport for complete instructions.

Additional Information: See Urine Preservatives-Collection and Transportation for 24-Hour Urine Specimens for multiple collections.

#### Specimen Minimum Volume:

0.3 mL

#### **Urine Preservative Collection Options:**

Note: The addition of preservative or application of temperature controls must occur within 4 hours of completion of the collection. Ambient (Plain) OK Refrigerate (Plain) Preferred Frozen (Plain) OK 50% Acetic Acid OK Boric Acid No Diazolidinyl Urea No 6M Hydrochloric Acid OK

6M Nitric Acid OK Sodium Carbonate No Thymol No

Toluene No

# **Test Definition: TLU**

Thallium, 24 Hour, Urine

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	

# Ask at Order Entry (AOE) Questions:

MAYO CLINIC LABORATORIES

Test ID	Question ID	Description	Туре	Reportable
TLU	TIME2	Collection Duration	Plain Text	Yes
TLU	VL12	Urine Volume	Plain Text	Yes

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
8603	Thallium, 24 Hr, U	Numeric	mcg/24 h	5746-3
TIME2	Collection Duration	Alphanumeric		13362-9
VL12	Urine Volume	Alphanumeric		3167-4

LOINC and CPT codes are provided by the performing laboratory.

# **Supplemental Report:**

No

## **CPT Code Information:**

83018

## **Reference Values:**

0-17 years: Not established > or =18 years: <2 mcg/24 hours