

Test Definition: HBAGP

Hepatitis B Virus Surface Antigen Prenatal, Serum

Reporting Title: HBs Antigen Prenatal, S

Performing Location: Rochester

Ordering Guidance:

This test should not be used to test symptomatic individuals who may or may not have risk factors for hepatitis B virus (HBV) infection. For testing such individuals, order HBAG / Hepatitis B Virus Surface Antigen, Serum.

This test should not be used to screen or test asymptomatic, nonpregnant individuals with or without risk factors for HBV infection. For testing such patients, order HBGSN / Hepatitis B Virus Surface Antigen Screen, Serum.

This test is not intended for testing cadaver or grossly hemolyzed specimens. For testing such patients, order HBGCD / Hepatitis B Surface Antigen for Cadaveric or Hemolyzed Specimens, Serum, which is US Food and Drug Administration-approved for testing on these sources.

Additional Testing Requirements:

Testing for acute hepatitis B virus (HBV) infection should also include HBIM / Hepatitis B Virus Core IgM Antibody, Serum, as during the acute HBV infection "window period," hepatitis B virus surface (HBs) antigen and HBs antibody may not be detected.

Necessary Information:

- 1. Date of collection is required.
- 2. Indicate if specimens are from autopsy/cadaver or hemolyzed sources so that the proper US Food and Drug Administration-licensed assay can be performed.

Specimen Requirements:

Supplies: Sarstedt Aliquot Tube, 5 mL (T914) Collection Container/Tube: Serum gel Submission Container/Tube: Plastic vial

Specimen Volume: 0.9 mL Collection Instructions:

- 1. Centrifuge blood collection tube per manufacturer's instructions (eg, centrifuge and aliquot within 2 hours of collection for BD Vacutainer tubes).
- 2. Aliquot serum into a plastic vial.

Specimen Minimum Volume:

0.7 mL

Forms:

If not ordering electronically, complete, print, and send 1 of the following with the specimen:

-Infectious Disease Serology Test Request (T916)



Test Definition: HBAGP

Hepatitis B Virus Surface Antigen Prenatal, Serum

-Gastroenterology and Hepatology Test Request (T728)

Specimen Type	Temperature	Time	Special Container
Serum SST	Frozen (preferred)	90 days	
	Refrigerated	6 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
HBSAP	HBs Antigen Prenatal, S	Alphanumeric		5196-1

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

87340 87341 (if appropriate)

Reflex Tests:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
HBNTP	HBs Ag Confirmation Prenatal, S			No	No

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
HBNTP	HBNTP	HBs Ag Confirmation Prenatal, S	Alphanumeric		7905-3



Test Definition: HBAGP

Hepatitis B Virus Surface Antigen Prenatal, Serum

Reference Values:

Negative

See Viral Hepatitis Serologic Profiles.