

---

**Reporting Title:** Fluticasone 17-B Carboxylic Acid, U  
**Performing Location:** Rochester**Specimen Requirements:**

Collection Container/Tube: Clean, plastic urine collection container

Submission Container/Tube: Plastic, 10-mL urine tube (T068)

Specimen Volume: 5 mL

Collection Instructions:

1. Collect a random urine specimen.
2. No preservative.

**Specimen Minimum Volume:**

0.6 mL

Specimen Type	Temperature	Time	Special Container
Urine	Frozen (preferred)	14 days	
	Refrigerated	72 hours	
	Ambient	24 hours	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
89739	Fluticasone 17-B Carboxylic Acid, U	Numeric	pg/mL	46952-8

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

80299

---

**Reference Values:**

Negative

Cutoff concentration: 10 pg/mL

Values for normal patients not taking fluticasone propionate should be less than the cutoff concentration (detection limit).