

Reporting Title: Cyclobenzaprine (Flexeril)**Performing Location:** Medtox Laboratories, Inc.**Specimen Requirements:**

Submit only 1 of the following specimens:

Plasma

Specimen Type: Plasma

Container/Tube: Green Top

Specimen Volume: 2 mL

Collection Instructions: Draw blood in a green-top (sodium heparin) tube, plasma gel tube is not acceptable. Spin down and send 2 mL of sodium heparin plasma refrigerated in a plastic vial.

Serum

Specimen Type: Serum

Container/Tube: Red

Specimen Volume: 2 mL

Collection Instructions: Draw blood in a plain, red-top tube, serum gel tube is not acceptable. Spin down and send 2 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume:

0.4 mL

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	7 days	
	Frozen	30 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
Z1130	Cyclobenzaprine (Flexeril)	Alphanumeric		6797-5

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80369

Reference Values:

Reference Range: 10 - 30 ng/mL