

Reporting Title: Cholecystokinin (CCK)**Performing Location:** Inter Science Institute**Specimen Requirements:**

Patient Preparation: The patient should fast for 10 to 12 hours prior to collection of specimens. Antacid medications and medications that affect intestinal motility should be discontinued, if possible, for at least 48 hours prior to collection of specimens.

Supplies: MCL Supply T125

Container/Tube: Special tube containing G.I. preservative (MCL Supply T125)

Specimen Volume: 3 mL

Collection Instructions:

1. Patient should fast for 10 to 12 hours prior to collection.
2. Collect 10 mL of blood in special GI preservative tube.
3. Specimen should be separated immediately, and plasma frozen as soon as possible and shipped.

Specimen Minimum Volume:

1 mL

Specimen Type	Temperature	Time	Special Container
GI Plasma	Frozen	180 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
Z0927	Cholecystokinin (CCK)	Alphanumeric	pg/mL	2081-8

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83519

Reference Values:

Up to 80 pg/mL