

Reporting Title: Angiotensin I, P**Performing Location:** Inter Science Institute**Specimen Requirements:**

Patient preparation: Patient should be on a normal sodium diet, 110 mEq of sodium. Patient should be in a recumbent posture for at least 30 minutes prior to collection of specimen. Diuretics, mineralocorticoids, glucocorticoids, estrogens, oral contraceptives, and ACTH medications and sodium, potassium, and posture all affect Angiotensin levels.

Specimen Type: Plasma

Collection Container/Tube: EDTA

Submission Container/Tube: Plastic vial

Specimen Volume: 3 mL

Collection Instructions:

Draw blood in a lavender-top (EDTA) tube(s).

Separate plasma from cells immediately after collection and send 3 mL of EDTA plasma frozen in plastic vial.

Specimen Minimum Volume:

1 mL

| Specimen Type | Temperature | Time | Special Container |
|---------------|-------------|---------|-------------------|
| Plasma EDTA | Frozen | 30 days | |

Result Codes:

| Result ID | Reporting Name | Type | Unit | LOINC® |
|-----------|------------------|--------------|-------|--------|
| Z0939 | Angiotensin I, P | Alphanumeric | pg/mL | 1858-0 |

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83520

Reference Values:

Up to 25 pg/mL