

Reporting Title: Styrene, Blood**Performing Location:** Medtox Laboratories, Inc.**Specimen Requirements:**

Collect 2 tubes green-top (sodium heparin) whole blood. Send 20 mL sodium heparin whole blood refrigerated. Collect specimen at end of shift or prior to next shift. Tubes should be filled to prevent loss of volatile compound into headspace.

Specimen Minimum Volume:

2.5 mL

Specimen Type	Temperature	Time	Special Container
WB Sodium Heparin	Refrigerated (preferred)	14 days	
	Frozen	180 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
Z1179	Styrene	Alphanumeric		8243-8

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

84600

Reference Values:

Normal (Unexposed population):
None detected

Exposed:
Biological Exposure Index (BEI):
0.55 mg/L (end of shift)

0.02 mg/L (prior to next shift)

Toxic:
Not established