

## **Test Definition: FSOTA**

Sotalol (Betapace)

Reporting Title: Sotalol (Betapace)

**Performing Location:** Medtox Laboratories, Inc.

### **Specimen Requirements:**

Submit only 1 of the following specimens:

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), plasma gel tube is not acceptable. Spin down and send 2 mL sodium heparin plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain red-top tube(s), serum gel tube is not acceptable. Spin down and send 2 mL of serum refrigerated in a plastic vial.

#### **Specimen Minimum Volume:**

0.3 mL

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	14 days	
	Frozen	180 days	
	Ambient	72 hours	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z1192	Sotalol	Alphanumeric		12416-4

LOINC and CPT codes are provided by the performing laboratory.

#### **Supplemental Report:**

No

#### **CPT Code Information:**

80299



# **Test Definition: FSOTA**

Sotalol (Betapace)

#### **Reference Values:**

Reference Range: 500 - 4000 ng/mL

Serum Sotalol concentrations producing beta-blockade:

500 - 4000 ng/mL

Toxic range has not been established.