

Reporting Title: Sertraline (Zoloft)**Performing Location:** Medtox Laboratories, Inc.**Specimen Requirements:**

Submit only 1 of the following specimens:

Plasma

Specimen Type: Plasma

Container/Tube: Green Top

Specimen Volume: 2 mL

Collection Instructions: Draw blood in a green-top (sodium heparin) tube, plasma gel tube is not acceptable. Spin down and send 2 mL of sodium heparin plasma refrigerated in a plastic vial.

Serum

Specimen Type: Serum

Container/Tube: Red

Specimen Volume: 2 mL

Collection Instructions: Draw blood in a plain, red-top tube, serum gel tube is not acceptable. Spin down and send 2 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume:

1.2 mL

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Varies | Refrigerated (preferred) | 7 days | |
| | Frozen | 180 days | |
| | Ambient | 72 hours | |

Result Codes:

| Result ID | Reporting Name | Type | Unit | LOINC® |
|-----------|---------------------|--------------|------|--------|
| Z2298 | Sertraline | Alphanumeric | | 6906-2 |
| Z2299 | Desmethylsertraline | Alphanumeric | | 6897-3 |

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80332

Reference Values:

Sertraline:

Reference Range: 30 - 200 ng/mL

Report Limit 10 ng/mL

Desmethylsertraline: ng/mL

No reference range provided

The stated reference range is the range of observed steady-state concentrations in individuals receiving therapeutic dosage regimens of sertraline. This is not a defined therapeutic range.

Report Limit 10 ng/mL