

**Reporting Title:** GlycoMark**Performing Location:** Esoterix Endocrinology**Specimen Requirements:**

Submit only 1 of the following specimens:

**Serum**

Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 1 mL of serum ambient in a plastic vial.

**Plasma**

Draw blood in a lavender-top (EDTA) tube(s). Spin down and send 1 mL of EDTA plasma ambient in a plastic vial.

**Specimen Minimum Volume:**

0.75 mL Note: This volume does not allow for repeat testing.

Specimen Type	Temperature	Time	Special Container
Varies	Ambient (preferred)	7 days	
	Frozen	14 days	
	Refrigerated	14 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
Z0275	GlycoMark	Alphanumeric		53835-5

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

84378

---

**Reference Values:**

Age	Range
<18 y	Not Established
Adult Males	10.7-32.0
Adult Females	6.8-29.3
Glycemic control goal for diabetic patients: >10	

GlycoMark is intended for use with managing glycemic control in diabetic patients. A low result corresponds to high glucose peaks.

1, 5-AG blood levels can be affected by clinical conditions or medications.